


# Class 6 Indicator

**Valisafe**

clearly a better indicator

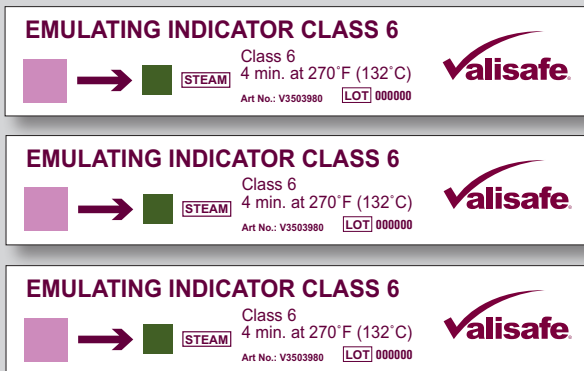

Easy to use

- Conforms to ANSI / AAMI / ISO 11140-1 Class 6
- Class 6 emulating indicator
- Specifically calibrated to 4 min at 132°C (270°F) cycle
- Monitors critical parameters
  - Time
  - Temperature
  - Steam
- For use in every tray or item
- Clear and abrupt colour change
- Easy to interpret - easy to record


## Valisafe Emulating Indicator Class 6, 4 minutes at 132°C

The Valisafe emulating Indicator provides an integrated response to steam sterilization in pre vacuum sterilizers operating at 132°C (270°F) and 135°C. The Emulator gives a clear visible confirmation that all critical parameters (time, steam quality and temperature) have been met at the location of the indicator within the sterilizer chamber.

### Order Details

V3503980      Emulating Indicator Class 6, 4 minutes at 132°C  
Bag of 250 indicators, 20 bags per case




**EMULATING INDICATOR CLASS 6**



Class 6  
4 min. at 270°F (132°C)

Art No.: V3503980    **LOT** 000000

**STEAM**

<b>Name:</b>	Class 6 Valisafe Emulating Indicator
<b>Classification:</b>	ANSI / AAMI / ISO 11140-1 Class 6
<b>Operating temp:</b>	132°C (270°F) or 135°C
<b>Time:</b>	4 minutes at 132°C and 3 minutes at 135°C
<b>Storage conditions:</b>	50-100°F (10-38°C)
<b>RH:</b>	10%-90%
<b>Lot no:</b>	yes
<b>Prod date:</b>	yes on box
<b>Exp date:</b>	3 years
<b>Correct symbols:</b>	In accordance with ANSI / AAMI / ISO 11140-1 Class 6
<b>Indicator:</b>	Class 6
<b>Disp/single use:</b>	yes
<b>Extra information:</b>	
Coated	yes
Non toxic/leadfree	yes
Lot no	yes
<b>Recordable information:</b>	
Date	yes
Operator	yes
Pass/fail result	yes
Colour change	pink to green

**Before**  
Original indicator



**After**  
After total process

